ZHISHAN 10217 FFP2 NR CE2163

CE2163 FILTERING HALF MASKS FOR PROTECTION AGAINST PARTICLES

Filtering half mask FFP2 NR ZHISHAN type 10217-EN149:2001+A1:2009 - Regulation (EU) 2016/425

USER INSTRUCTION LEAFLET

Please read this brochure carefully before using the product to avoid misuse of the product.

Failure to follow instructions and use limitations could reduce the effectiveness of the particulate filter half mask and could lead to illness, injury or even death.

Intended use of filtering half masks for protection against particles:

- The filtering half mask for protection against particles is a complete respiratory protection equipment and it is intended to protect the user against the harmful effect of air pollution in the form of solid and / or liquid particles that form aerosols (dust, smoke and mist)...

Check before use: The filtering half mask for protection against particle should be checked before each use to make sure it is not damaged.

Checking the fit: To check the fit of the filtering half mask for protection against particles, place both hands on the mask and inhale deeply...

Use only one shift (NR): The filtering half mask for protection against particle should not be used for more than one work shift.

Warning: The user must be trained to be able to correctly select and use the mask in accordance with the applicable health and safety regulations.

Storage conditions: Store the product at an ambient temperature of -30°C +30°C and a maximum humidity of 80%.

See manufacturer information: Do not re-use. Do not make modifications to the mask.

Conformity symbol: SYMBOL WITH THE NUMBER OF THE LABORATORY THAT CONDUCTED AN EXAMINATION

Manufacturer Shenzhen Zhishan Medical Co., Ltd Baolong street 5, Longgang, Shenzhen, China

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Fabrikant: Shenzhen Zhishan Medical Co., Ltd Baolong street 5, Longgang, Shenzhen, China